

Quantiferon TB Testing - One Tube Collection

Getting Started:

Please contact your local Labcorp representative to set-up a Labcorp account, begin ordering and receiving results and/or order initial requisition forms and supplies.

Test Name: Quantiferon –TB Gold Plus

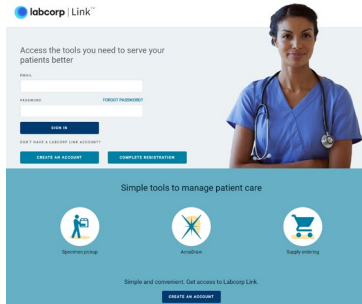
Labcorp Test Number: 182913




Expected Turn-Around Time: 2-5 Days

Collection Instructions: Fill a lithium heparin blood collection tube (minimum volume 6 mL) and gently mix by inverting the tube several times to dissolve the heparin. Transport to the lab refrigerated.

Storage Instructions: After collection blood must be refrigerated within 3 hours. Refrigerated Sample is stable up to 48 hours.

Supply Ordering: www.labcorplink.com



<p>PeopleSoft/Part #: 39931</p> <p>Name: Tube Green Top Lithium Heparin 6.0ml, 1 Tube, [367886]</p> 	<p>PeopleSoft/Part #: 38800</p> <p>Name: 4900 .Gel Propak 8 Oz</p> 	<p>PeopleSoft/Part #: 110475</p> <p>Name: Zlab-stable69 .Refrigerated Specimen Bag 6x9 3-Wall Tearzone</p> 
---	--	--

Customer Support: 800-762-4344 Option 2 then...			
Results	Option 1	Computer Support	Option 5
Specimen Pick-Up	Option 3	Supplies	Option 6