

## ESO Solutions – Zoll Cloud/X Series Configuration

### Zoll Online Setup

1. If you do not already have one, you will need to create a Zoll Online account.

<https://www.zollonline.com>

2. After your account is setup, you will need to contact Zoll Support to have them enable **Case Review Basic**.
3. Create a Data Access Key by clicking on “View” under CaseReview Basic then Settings→New access key.

### My Products

The screenshot displays the 'My Products' section of the Zoll Online interface. It features four product cards arranged in a grid. Each card includes an icon, a title, a brief description, and a 'View' button. A red arrow points to the 'View' button under the 'CaseReview Basic' card.

- 12-Lead**: Receive and manage 12-lead ECGs from just about anywhere. Includes links for [User Guide](#) and [Admin Guide](#).
- CaseReview Basic**: Clinical Case Management System. Includes a [Help](#) link.
- Checklist**: Create and run through your agency's pre-trip vehicle checks.
- Road Safety**: Improve the safety of your fleet by monitoring driver behavior.

4. Give the access key a name and description assign a password and check “enabled”. Please note you will be entering this password in ESO Admin and in your X Series Monitors.

Administer access keys

Name

Description

Password

Access key -----

Alternate ID -----

Enabled


5. Once you click “Save” you will be taken to the main screen and will be able to see the “Alternate ID” and “Password”.

The screenshot shows the ZOLL online settings interface. The browser address bar displays <https://ez.zollonline.com/Settings/EditSettings>. The navigation menu includes Home, Admin, Company, and Logout. The main content area is divided into several sections: CPR Quality Settings, User Permissions, CaseReview Importer, Data Access Keys, and File archive. The Data Access Keys section is highlighted with a red box and contains a table with the following data:

Name	Description	Access key	Alternate ID	Password	Enabled	
Case Review	Case Review	803a0879-5795-e611-80d3-005056a32e8e	1610181722436	1234	<input checked="" type="checkbox"/>	Edit

## X Series Monitor Setup

1. On the X Series Monitor go to Supervisor→Communications→Disclosure Log  
Server DNS Name: DXSVC.ZOLLONLINE.COM  
Port: 443  
Password: Enter your newly created password.  
Customer ID: Enter your newly created "Alternate ID".

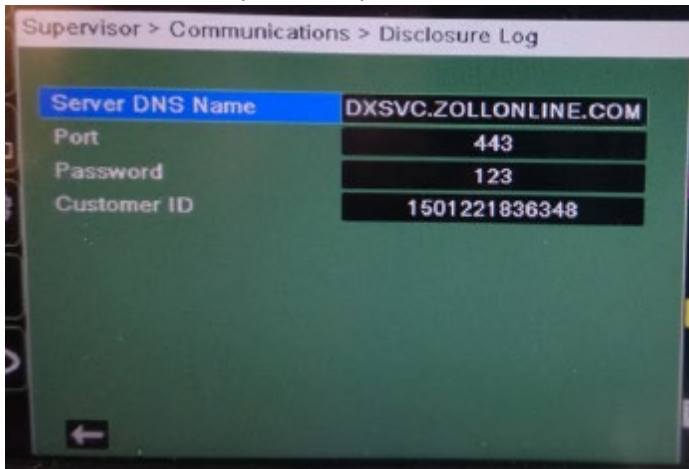

## ESO Admin Setup

1. Log into <https://www.esosuite.net> and navigate to Admin→EHR→ECG Monitor Integration
2. Enter your Alternate ID and Password in the fields indicated below:

The screenshot shows a web form titled "Edit ZOLL Cloud Credentials". At the top, there is a "Password" label with a red "REQUIRED" tag. The form contains two input fields, both highlighted in yellow:

- User Name:** A text input field containing "Alternate ID".
- Password:** A password input field containing "Password".

At the bottom right of the form, there are two buttons: a grey "Cancel" button and a green "Done" button.

3. Next, navigate to Admin→EHR→Devices, click “Add” and enter in the information indicated below FOR EACH X Series Monitor (NOTE: “Defibrillator-Manual” and “ECG-12 Lead or Greater” are the only 2 items that need to be selected):

### New Cardiac Monitor or Device

Name  
**Monitor name** ←

Type  
**Defibrillator-Manual, ECG-12 Lead or Greater** ←

Serial  
Zoll Serial number ←

Manufacturer

Model

Purchase Date  
mm/dd/yyyy

Cancel Done