

GIVE A HAND TO PREVENT NEEDLE STICKS

Safety needles are activated with the same hand used to administer the shot.

When safety needles are used correctly and immediately disposed of in a sharps container after use, needle sticks can be eliminated.



Eclipse

- Use one hand to press the safety cover on a hard surface such as the edge of a table
- Confirm the needle is fully engaged in the safety cover
- Dispose of in a sharps container



Vanishpoint

- Use one hand to fully depress the plunger squeezing through the resistance point to deliver full medication dose
- Needle automatically retracts into the syringe barrel
- Dispose of in a sharps container



Safetyglide

- Use one hand to push the lever arm completely forward with your thumb until the needle tip is completely covered
- Dispose of in a sharps container



Longevity Retractable

- Pull the plunger back to draw the needle into the syringe
- Once the plunger is as far back as possible, detach the plunger from the syringe leaving the needle
- Dispose of in a sharps container